PM 211AA

Alternating Pressure Relief System

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German

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IMPORTANT SAFEGUARDS

READ ALL INSTRUCTIONS BEFORE USING

DANGER - To reduce the risk of electrocution:

1. Always unplug this product immediately after using.
2. Do not use while bathing.
3. Do not place or store this product where it can fall or be pulled into a tub or sink.
4. Do not place in or drop into water or other liquid.
5. Do not reach for a product that has fallen into water. Unplug immediately.

WARNING -To reduce the risk of burns, electrocution, fire, or injury to persons:

1. This product should never be left unattended when plugged in.
2. Close supervision is necessary when this product is used by, on, or near children or invalids.
3. Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer
4. Never operate this product if it has a damaged cord or plug, if it is not working properly, or if it has been dropped, damaged, or dropped into water. Return the product to a service center for examination and repair.
5. Keep the cord away from heated surfaces.
6. Never block the air openings of this product or place it on a soft surface, such as a bed or couch, where their openings may be blocked. Keep the air openings free of lint, hairs, and other similar particles.
7. Never drop or insert any object into any opening or hose.
8. Connect this product to a properly grounded outlet only.

NOTE, CAUTION AND WARNING STATEMENTS:

NOTE -Indicate some tips.

CAUTION - Indicate correct operating or maintenance procedures in order to prevent damage to or destruction of the equipment or other property

WARNING - Calls attention to a potential danger that requires correct procedures or practices in order to prevent personal injury.

SYMBOLS

Manufacturer

Authorized representative in the European community

Attention, should read the instructions

"BF" symbol, indicate this product is according to the degree of protecting against electric shock for type BF equipment.

Grounding terminal (for 230V System)

Functional earth(for UL only)

Consult operating instructions

Class II (for UL only)

Disposal of Electrical & Electronic Equipment (WEEE):

This product should be handed over to an applicable collection point for the recycling of electrical and electronic equipment. For more detailed information about the recycling of this product, please contact your local city office, household waste disposal service or the retail store where you purchased this product.

1. INTRODUCTION

This manual should be used for initial set up of the system and for reference purposes.


1.1 General Information

The system is a high quality and affordable mattress system suitable for treatment and prevention of pressure ulcers.

The system has been tested and successfully approved to the following standards:

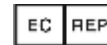EN 60601-1
EN 60601-1-2
EN 55011 Class B
IEC61000-3-2
IEC 61000-3-3

APEX MEDICAL S.L.

Máximo Aguirre 18 Bis, 8ª planta,
48011 Bilbao. Vizcaya. SPAIN

APEX MEDICAL CORP.

9, Min Sheng St., Tu-Cheng,
Taipei County, 236, Taiwan

1.2 Intended Use

This product is intended to help and reduce the incidence of pressure ulcers while optimizing patient comfort. It also provides following purposes:

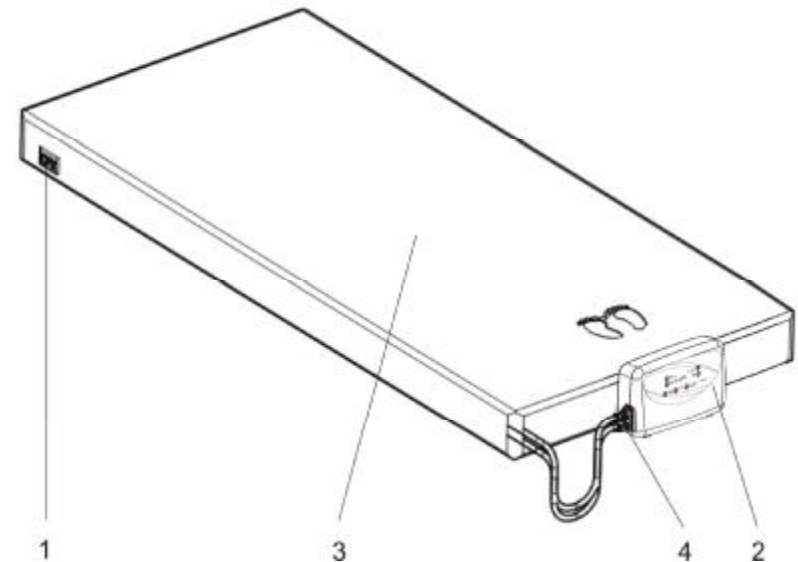
- Individual home care setting and long term care of whom suffering from pressure ulcer.
- Pain management as prescribed by a physician

 **NOTE** - Equipment not suitable for use in the presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide.

2. PRODUCT DESCRIPTION


Unpack the box to check for any damage which may have occurred during shipment. If there are damages, please contact your dealer immediately.

2.1 Pump and Mattress System


1. CPR
2. Pump unit
3. Mattress System
4. Quick Connector

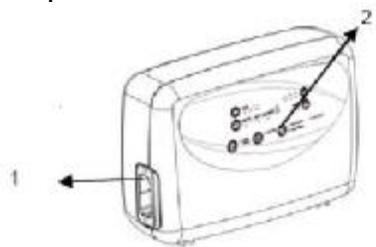2.2 Pump Unit

Front

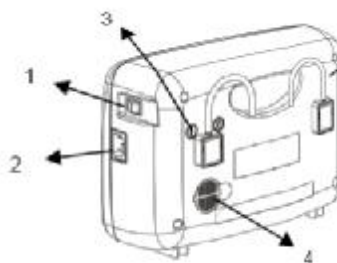
- 1.Quick Connectors
- 2.Front Panel

Rear

- 1.Power Switch
- 2.Power Cord
- 3.Air Filter
- 4. Fuses
- 5.Mounting Brackets

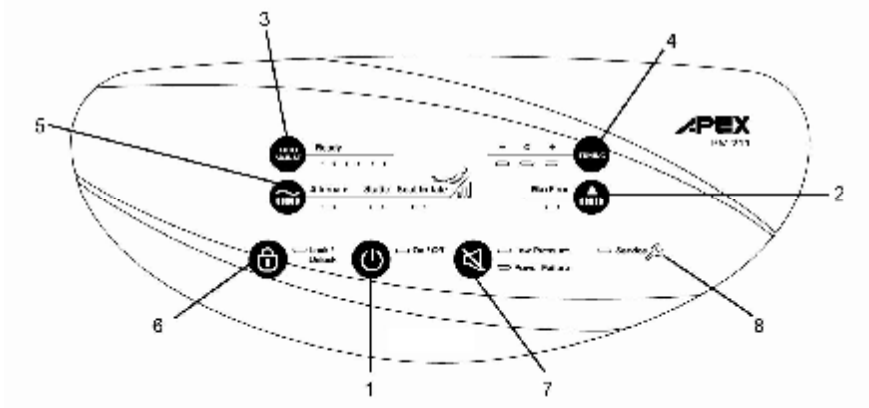2.3 Front panel

1. **POWER**

Enable / Disable the control unit

⚠ NOTE: Check and ensure the main power switch, found from the side of pump, is ON.

2. MAXFIRM

The Maxfirm mode will be forced to execute every time the power switch is turned on. It insures the pump is able to reach its maximum operating pressure. This mode will last for 20 minutes and then it will return to previous setting automatically. To disable this function, press Maxfirm button again.

3. Pressure Adjust

The function can automatically adjust the mattress pressure based on the patient's weight. To remind user the process is on-going, the function LED lights will be periodically flashed from left to right until the Ready LED turn on permanently (the process is completed). There are three conditions will trigger this function to ensure the appropriate pressure setting for the patient.

- A) Press the Auto-Adjust button for 2 seconds, system will automatically re-adjust the pressure setting.
- B) The system will automatically start Auto-Adjust immediately after the initial mattress inflation is completed.
- C) While the system detects a significant change in patient's weight on the mattress (e.g. patient Ingress) for a period of time, the system will automatically run the function again.

While the process is completed, system will return to previous setting automatically. To disable this function, press Auto-Adjust button again.

NOTE: During the Auto-Adjust operation, it is normal that system will go through a series of inflation and deflation.

4. TUNING

The Tuning function is to fine-tune the system auto-adjusted preset pressure. Three Tuning levels are available. "0" is system auto-adjusted preset pressure. "+" is one level higher than preset pressure, and "-" is one level lower than preset pressure.

NOTE: The Tuning function is only available in alternate and static mode.

5. Therapy.

The Therapy button allows you to choose one of the therapy modes. There are four selectable therapy modes available. Continually press Therapy button to select a desired mode:

- A. Alternate
The mattress will alternate every 10 minutes under this therapy mode.
- B. Static
The pressure inside of air tubes will be adjusted to the same pressure.
- C. Seat Inflation
The seat inflation features additional supports to the patient during upright position.

6. Panel Lock

The panel lock feature will lock the panel with green LED light on to prevent accident from changing the setting during normal operation. To lock and unlock, simply by pressing the Panel LOCK button for 2 second.

7. Alarm Mute

Press alarm mute button to temporary suspend the low-pressure alarm (LED light and buzzer). Should the situation not resolved within 5 mins, the alarm shall resume to notify the patient.

- A. PFA (Power Failure Alarm)
During power failure situation, the Power Failure LED light will lit on with buzzer. By pressing the mute button to disable both buzzer and LED.
- B. Low Pressure Indicator
When low pressure LED lights up, the pressure inside of air mattress is below normal. Please refer to troubleshooting.

8. Service

This feature will lit during mechanical failure situation. User can notify the technician for repair.

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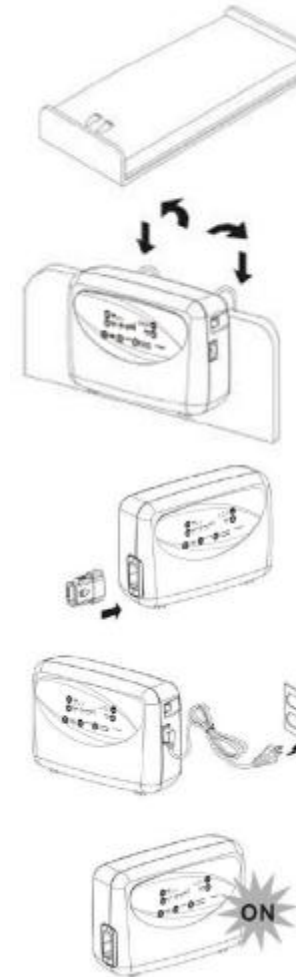
3. INSTALLATION

Unpack the box and then inspect for any damage which may have occurred during shipment. If there are any damages, please contact your dealer immediately.

3.1 Pump & Mattress Installation

1. Place the mattress or pad on top of the bed frame. Please note for the foot end.
2. Hang the pump onto bed rail (foot-end), and adjust hangers to best upright position of the pump, or place the pump on a flat surface.
3. Connect air hose connectors from air mattress to the pump unit. When a "click" sound is felt or heard, the connection is completed and secured.

NOTE: Check and ensure the air hoses are not kinked or tucked under mattress.

4. Plug the power cord into electrical outlet.

NOTE:

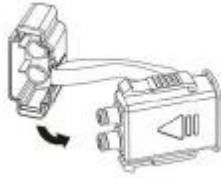
1. Make sure the pump unit is suitable for the local power voltage.
2. The plug is also served to disconnect the device.

CAUTION: The pump can only be applied to the mattress recommended by the manufacturer. Do not use it for any other purpose. (Applied part: air mattress)

5. Then turn the main power switch to ON position.

⚠ NOTE: (For non low air loss model only) During power outage situation, the caregiver can use the transport cap (for non low air loss model only) to keep the air inside the air cells inflated by capping the quick connectors. (Shown above)

Grounding:

Before any connection to the output connectors is made, the unit shall be connected to a protective earth conductor via the three-core main cable; the mains plug shall be inserted only into a socket outlet provided with a protective earth contact. The protective action shall not be negated by the user of an extension cord without protective conductor.

Several installation tips are listed below:

After installation, the extra length of the power cord, if any, should be neatly arranged to avoid any tripping accidents. The EQUIPMENT should be firmly placed at position where users/doctors can access easily.

⚠ NOTE - Do not fold, crease or stack the mattresses. Avoid direct sunshine.

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4. OPERATION

 **NOTE** - Always read the operating instruction before use.

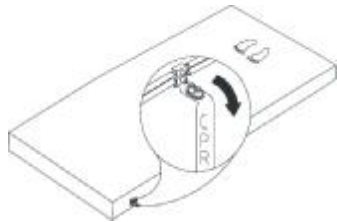
4.1 General operation

1. Switch on the main power switch on the side of the pump.
2. The pump unit is to deliver air into mattress, wait at least 20 minutes for mattress to be inflated.
3. The low-pressure indicator (yellow LED) will light up when the mattress is not fully inflated.
4. When the appropriate pressure is reached, the low-pressure indicator (yellow LED) will go off and pump will enter the alternating mode.
5. Alternatively, you can also preset the device by press THERAPY, COMFORT and CYCLE TIME button during initial inflation (Autofirm process). The system will automatically operate the intended settings immediately after the Autofirm process is completed. (Refer to section 2.3.3)
6. According to the weight and height of the patient, adjust the pressure setting to the most comfortable level without bottoming out, then the pressure in mattress will slowly increase to the intended value after the air mattress is ready to use.

 **NOTE:** Every time when mattress is first setup for use, it is recommended that the pressure levels to be set to "Max" for the quickest inflation. User can then adjust air mattress to the desired softness afterward.

4.2 CPR

When there is an emergency to perform CPR on the patient, quickly pull the CPR valves to release air from mattress. The CPR valves are located at the head-end, right-hand-side of the mattress. (Only available for air cell types mattress) The quick connector on the pump unit can be disconnected for even faster deflation.

4.3 PRESSURE FINE-TUNE

Depending on the desired preference, user can adjust mattress pressure slightly by pressing Tuning button.

⚠ NOTE - Check to see if the suitable pressure is selected by sliding one hand between the deflated air cells and the patient to feel's patient's buttock. Users should be able to feel the minimum contact.

If the pump unit is equipped without alarm buzzer, the Low Pressure indicator will light up until the low-pressure fault condition is resolved.

4.4 Low Pressure Function

When an abnormal low pressure is occurred, the Low Pressure indicator (yellow LED) will light up. Check that connections are correctly made and that they are correctly installed as per installation instructions.

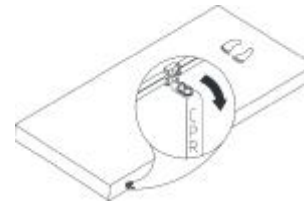
⚠ NOTE - If the pressure level is consistently low, check for any leakage (tubes or connecting hoses). If necessary, replace any damaged tubes or hoses. Or contact local qualified dealer for repair.

If the pump unit is equipped with alarm buzzer, the alarm will buzz and the Low Pressure indicator light up when air pressure is below normal. To mute the alarm, simply press the Alarm Reset button on the side. However, the low pressure indicator remains until the low-pressure fault condition is solved.

4.5 Static Mode

Press the THERAPY button to suspend the alternating mode of mattress. By pressing this button again, the mattress will be back to alternating therapy mode.

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4.6 Alarm Mute

When low-pressure situation occurred, both the LED light and buzzer will sound off to warn the patient. By pressing the button, it will temporary mute the buzzer so the care giver may check for possible air leaks. Should the situation not resolved within 5 minutes; the alarm will resume beeping.

5. CLEANING

It is important to follow the cleaning procedures before using the equipment on human bodies; otherwise, patients and/or doctors may have the possibility of getting infection.

Wipe the pump unit with a damp cloth and a mild detergent, and keep it away from dust. If other detergent is used, choose one that will have no chemical effects on the surface of the plastic case of the pump unit.

 **CAUTION-** Do not immerse or soak pump unit.

Wipe down the mattress with warm water containing a mild detergent. The cover may also be cleaned by using sodium hypochlorite diluted in water. All parts should be air dried thoroughly before use.

 **CAUTION-** Do not use phenolic-based product for cleaning.

 **CAUTION-** Dry the mattress on SUNLESS capitalized area after cleaning.

The carrying bag (if available) should be turned inside out and completely wiped down using the disinfectant solutions. Allow it to air-dry thoroughly. Once the inside is dry, turn it back and wipe down the outside of the bag with disinfectant solutions.

6. STORAGE

1. To store the mattress, lay the mattress out flat and upsides down.
2. Roll from the head end towards the foot end with CPR valve open.
3. Foot-end strap can then be stretched around the rolled mattress to prevent unrolling.

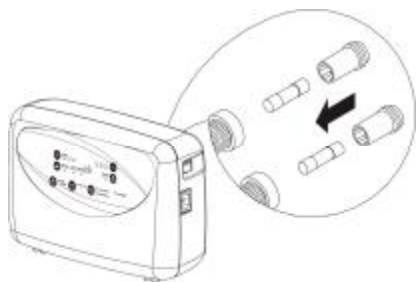
 **NOTE-** Do not fold, crease or stack the mattresses.

7. MAINTENANCE

7.1 General

1. Check main power cord and plug if there is abrasions or excessive wear.
2. Check mattress cover for signs of wear or damage. Ensure mattress cover and tubes are stubbed together correctly.
3. Check airflow from the quick connector. The airflow should alternate between each connector every half-cycle time if it's in alternating mode.
4. Check the air hoses if there is kink or breaks. For replacement, please contact our local dealers.

7.2 Fuse Replacement

1. Disconnect the plug from mains power when a blown fuse is suspected.
2. Remove the cover of the fuse holder by means of a small screwdriver.
3. Insert a new fuse of the correct rating in, and replace the cover of the fuse holder back. The fuse should be rated as T1A/250V type and VDE approved.

7.3 Air Filter Replacement

1. Loosen the screws from the air filter plate located at the back of pump.
2. The filter is reusable and can be washed gently with a mild detergent and water. Dry the filter before use.
3. Check and replace air filter regularly if environment is dirty.

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8. TECHNICAL SPECIFICATIONS:

| Item | | Specification |
|--|-------------|--|
| Power Supply (Note: See rating label on the product) | | AC 120V 50/60 Hz, 0.17A (for 120V system) AC 230V 50/60 Hz, 0.07A (for 230V system) |
| Fuse Rating | | T1A, 250V |
| Cycle time | | Four selectable |
| Dimension (L x W x H) | | 29.1 x 20 x 11.7 cm / 11.4" x 7.9" x 4.6" |
| Weight | | 2.2 Kg / 5 lb |
| Environment | Temperature | Operation: 10°C to 40°C (50°F to 104°F) Storage: -15°C to 50°C (5°F to 122°F) Shipping: -15°C to 70°C (5°F to 158°F) |
| | Humidity | Operation: 10% to 90% non-condensing Storage: 10% to 90% non-condensing Shipping: 10 % to 90% non-condensing |
| Classification | | Class II (for UL only) with functional earth (NOTE: 5)*, Type BF, IPX0 Applied Part: Air Mattress Not suitable for use in the presence of a flammable anesthetic mixture (No AP or APG protection) |
| Mattress | | Specification |
| Model | | 8" Mattress |
| Dimension (L x W x H) | | 200 x 90 x 20.3 cm / 78.7" x 35.4" x 8" 200 x 85 x 20.3 cm / 78.7" x 33.5" x 8" 200 x 80 x 20.3 cm / 78.7" x 31.5" x 8" |
| Weight | | 7.5 kg /16.5 lb |
| Pressure Range | | 15 - 40 mmHg |

- ⚠ NOTE:**
1. Consult the distributor or EU representative for further technical documents.
 2. The specification is also suitable for other areas operating with same power supply.
 3. Mattress dimension and weight is measured without foam cushion.
 4. The manufacturer reserves the right to modify the specification without notice.
 5. The earthing terminal is served as functional earth, not protective earth.

SAVE THESE INSTRUCTIONS

GROUNDING INSTRUCTIONS

This product should be grounded. In the event of an electrical short circuit grounding reduces the risk of electric shock by providing an escape wire for the electric current. This product is equipped with a cord having a grounding wire with a grounding plug. The plug must be plugged into an outlet that is properly installed and grounded.

DANGER - Improper use of the grounding plug can result in a risk of electric shock.

If repair or replacement of the cord or plug is necessary, do not connect the grounding wire to either flat blade terminal. The wire with insulation having an outer surface that is green with or without yellow stripes is the grounding wire.

NOTE - If the repair or replacement of the cord is necessary, please contact a qualified electrician or serviceman. To reduce the risk of electric shock, do not modify the cord or plug in any way.

Check with a qualified electrician or serviceman if the grounding instructions are not completely understood, or if in doubt as to whether the product is properly grounded.

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