



**Keystone Healthcare Supplies**



## Changing Message- Proximate Pro System

1. Press DN
2. "Delay Sound " screen Press DN to toggle between Off/Loud/Chirp (should be off)
3. Press Enter
4. "Alarm Sound " screen Press DN to toggle between Off/Loud/Chirp
5. Press Enter
6. "Msg, Ent to chg" screen
7. Press Enter
8. "Confirm Chg Msg" screen Press DN to toggle between Yes/No
9. Press Enter once you have selected Yes
10. Press Up and Dn buttons to adjust letters
11. Press Enter to select desired characters N.B. Press Menu to backspace.
12. Press down to select full stop (.) then press enter
13. Press Menu/Esc to exit to Main Screen

For further Assistance please do not hesitate to contact our office 5976 4750