



Keystone Healthcare Supplies

Proxi-Mate Pro Setup

1. Connect sensor matt to the Proxi-Mate box
2. Connect Proxi-Mate box to a power supply in the wall
3. Press and hold ENTER to wake Bed Sensor up from sleep mode
(The word "OFF" will now appear on proximate screen)
4. Place **bed sensor** under the patient first
 - Should only be under one layer of linen
 - Place under the shoulders for an early warning, or under the lower back for a late warning. Refer figure 1.
N.B. As the patient is getting out of bed, their shoulders break contact with the bed before their lower back
5. Press, **ENTER** (Bed sensor is now "ON" and active)
Pager will beep once and display the message "Status Update: "on""
6. If bed sensor ALARMS, press ENTER once to turn off/reset.
Wait 5 seconds, the Proximate box will stop alarming and your pager will beep once and display the message " Status Update: "Reset"
7. Reposition patient over bed sensor strap. Press ENTER to reactivate program to "ON". Your pager will beep once and display the message "Status Update: ON."

