



Keystone Healthcare Supplies



Silent Function – Proximate Pro

Placing the Proximate on silent means the proximate box will not alarm whilst the pager will continue to alarm.

Silencing the Proximate Box means the patient will not be disturbed however only the nurse carrying the pager will be notified if the patient breaks contact with the sensor matt

1. Press DN

2. "Delay Sound " screen

Press DN to toggle between Off/Loud/Chirp (should be off)

3. Press Enter

4. "Alarm Sound " screen

Press DN to toggle between Off/Loud/Chirp

5. Once you have selected your desired setting press enter.

(Select Off to silence the Proximate Box)

For further Assistance please do not hesitate to contact our office
5976 4750.