




## Proxi-Mate Standard Nurse Call Setup



1. Connect sensor matt to the Proxi-Mate box
2. Connect Proxi-Mate box to a power supply in the wall
3. Place sensor matt under the patient first - Should only be under one layer of linen
4. Press the power button (the two lights will turn on indicating the device is now active)
5. If bed sensor ALARMS (the two lights will be flashing), press the power button once to turn off the device (the two lights will turn off indicating the device is not active)
6. Cancel the nurse call
7. Reposition patient over bed sensor strap
8. Press the power button (the two lights will turn on indicating the device is now active)

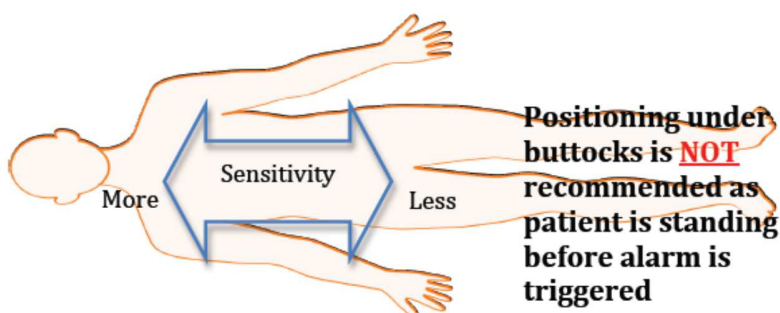

As the patient is getting out of bed, their shoulders break contact with the bed first.

Therefore place sensor matt under shoulders for an early warning, or under the lower back for a late warning.

