



Keystone Healthcare Supplies



Silent Function – Standard Proximate

Placing the Proximate on silent means the proximate box will not alarm whilst the pager will continue to alarm.

This means the patient will not be disturbed however only the nurse carrying the pager will be notified if the patient breaks contact with the sensor matt

1. Hold down the power button for 15 seconds until the proximate beeps once.
2. The Proximate Box will beep once
3. The Proximate Box is now silent
4. To turn on the Proximate Box's alarm sound, hold down the power button for 15 seconds until the proximate box beeps once

For further Assistance please do not hesitate to contact our office 5976 4750.